

Transcatheter Aortic Valve










Marker location and significance




Low radial strength
Provides
conformability to the
shape of the aorta


High hoop strength Ensures circular configuration of the bioprosthetic valve

High radial strength
Provides fixation to
the native aortic
annulus

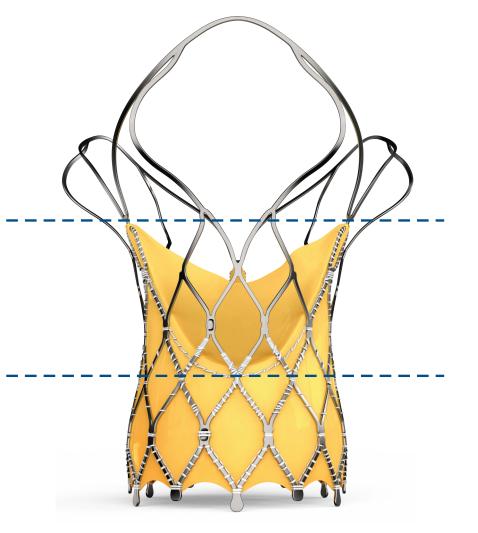


- First self-expanding TAVI device to have 2 rows of marker
 - First markers are located at Node 1
 - Second markers are located at Node 3
- First marker helps
 - In precise implantation of the valve at the targeted implantation zone
 - To ascertain the depth of implant
- Second marker indicates
 - When the THV leaflets are going to get deployed

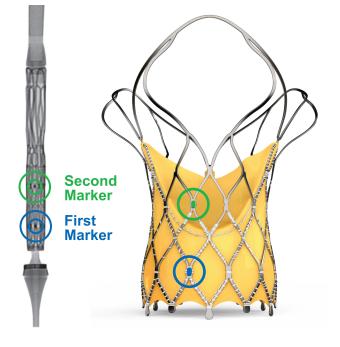

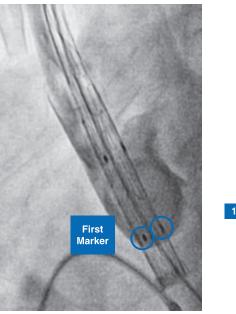



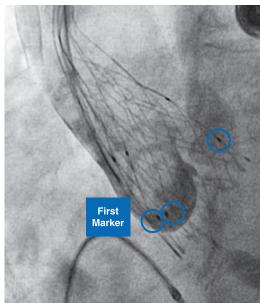

Case Example

Mid Frame

Inflow

Key Features



Bovine Pericardium



Self expandable Nitinol Frame

Conformable to native annulus

Three-tentacle design Less metal at outflow

Large Frame cells (≥15 Fr)
Facilitates easy access to


the coronary arteries

Extended Sealing skirt

Mitigates paravalvular leak


Non-flared inflow part


Reduces conduction abnormality

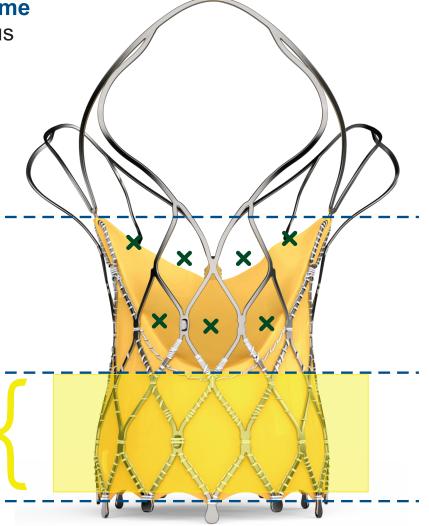

Each 'X' represents a single large cell

- Made from single bovine pericardium
- Bioprosthetic valve leaflets are supra-annular in position provides superior hemodynamics by providing larger effective orifice area and lower pressure gradient
- Supra-annular valve position helps maintain circular shape of the bioprosthetic valve even if the native annulus shape is elliptical
- Proprietary anti-calcification treatment

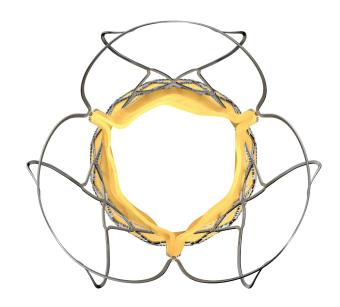


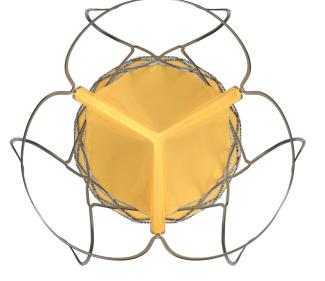






Capsule

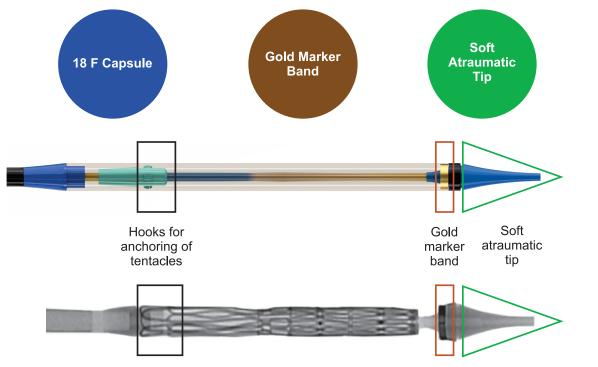

- Radiopaque soft atraumatic tip
- Sold marker band on the capsule helps in determining the location of sheath while release/recapture


Flexible Shaft and Stability layer

- Flexible shaft ensures flexibility of the overall system while navigating through the arch of aorta
- Stability layer enables stable catheter position during the deployment and aids in accurate valve implantation

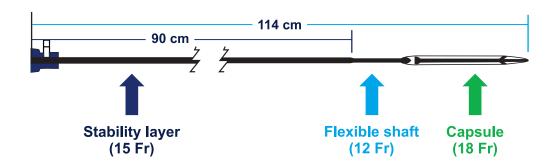

Control Handle

- > Three unique color-coded indicators to determine the deployment stage
 - Blue signifies fully encapsulated valve
 - Yellow signifies release of second marker
 - Orange signifies release of tentacles
- Blue Knob: Counterclockwise to Un-sheath (Release), Clockwise to Re-sheath (Capture)

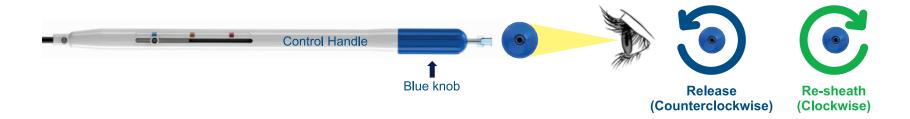

Flush Port

3 Dedicated Flush ports on the Control Handle



Sizes mentioned in "mm"




Recapturable up to 80% deployment





		Hydra 22	Hydra 26	Hydra 30
Diameter at Bottom	Α	22	26	30
Diameter at Top	В	39	43	47
Sealing Skirt Height	С	12	13	14
First Marker Crimped	X	8	9	10
First Marker	D	5	6	7
Second Marker	Е	15	18	21
Height	Н	55	53	51

Ø B

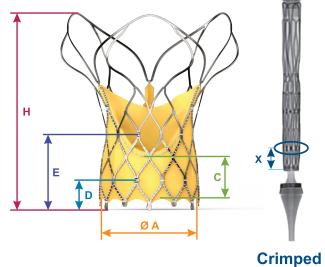

Intended Annular Treatment Range

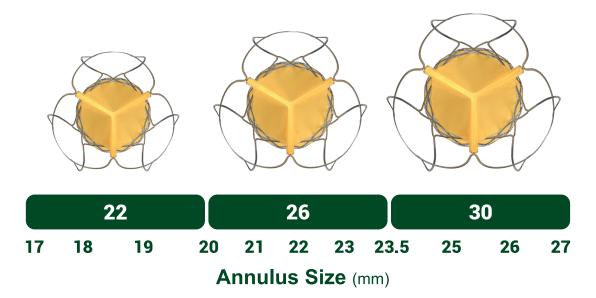

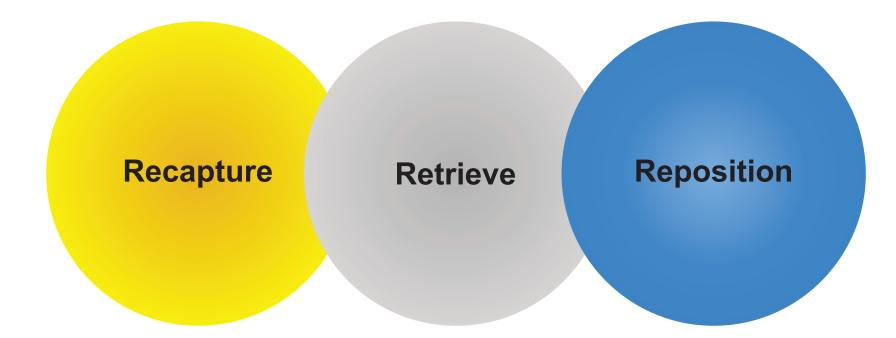

Hydra THV^{*} can be Re-captured up to 80% deployment

*Transcatheter Heart Valve

Clinical experience



Summary





Excellent Hemodynamics

Genesis study (n=40)*

0% moderate-to-severe paravalvular leakage

Mean aortic valve gradient, mmHg

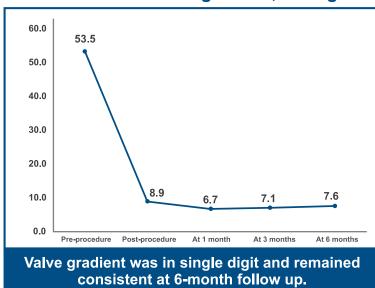


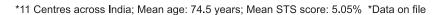

Ordering information

Reference Number	Size	Delivery Catheter System	Valve Loading system
HYDRA22	22 mm	HYDRA18F	HYDRA18VLS
HYDRA26	26 mm	HYDRA18F	HYDRA18VLS
HYDRA30	30 mm	HYDRA18F	HYDRA18VLS

Mean aortic valve area, cm²

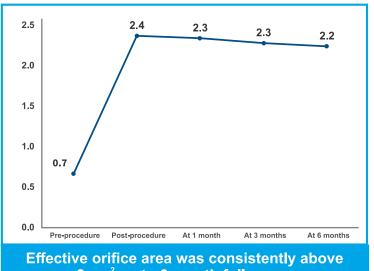

2 cm² up to 6-month follow up.

Valve Frame Varying radial force Supra-annular design Two set of markers Bovine on the frame for precise positioning pericardial tissue Highly flexible design and easy Anti-calcification navigation due to treatment less metal. Extended Sealing skirt Large open cells for future coronary access

	Hydra AVDC	Safety features		
٥	3 Unique color- coded indicators	•	Recapturable	
	for deployment guidance	•	Repositionable	
•	Ease of release of the valve	•	Retrievable	
•	One delivery system for three THV sizes			
•	18 Fr compatible			

www.smtpl.com

Head office - Surat

Sahajanand Medical Technologies Pvt. Ltd. Sahajanand Estate, Wakhariawadi, Near Dabholi, Ved Road, Surat - 395 004. GJ, India. Tel: +91 261 3060606 | +91 261 6112800 Fax: +91 261 3060607

E-mail: contact@sahmed.com

Poland

SMT Polonia sp. z o.o. Al. Grunwaldzka 411. 80-309 Gdańsk, Aurum, 5 pietro. Tel: +48 603 095 223 E-mail: marek.kurysz@smtpl.com

Corporate office - Mumbai

E-mail: contact@sahmed.com

Sahajanand Medical Technologies Pvt. Ltd. 221, C - Wing, Kankia Atrium, Andheri - Kurla Road AAI Colony, J B Nagar, Andheri East, Mumbai, Maharashtra 400059, MH. India. Tel: +91 22 3004 0000 | +91 22 6284 4000 Fax: +91 22 3004 0099

Ireland office

Sahajanand Medical Technologies Ireland Ltd. Ground Floor, Block 5, Galway Technology Park, Parkmore, Galway, H91 R9YR, Ireland, Tel: +353 9139 7472

E-mail: contact@sahmed.com

Russia

SMT CIS Building 1, 89 Krasnobogatyrskaya str. Moscow 107076, Russian Federation. Mrs.Maria Bokova Tel: +7 906 7368113 E-mail: maria.bokova@smtpl.com

Germany & Austria

SMT Germany GmbH Weiseler Strasse 16, 35510 Butzbach, Germany Tel: +496033.97 24 53-0 Fax: +496033.97 24 53-9

E-mail: customercare.dach@smtpl.com

Switzerland

SMT Switzerland AG Bundesstrasse 9, 6302 Zug. Tel: +41 (0) 41 71000 82 Fax: +41 (0) 41 71000 84 E-mail: customercare.dach@smtpl.com

Spain

SMT Iberia Leonardo Da Vinci, 22, 46980 Paterna Valencia SPAIN. Tel: +34 96 293 4934 Fax: +34 96 293 4933 E-mail: info.iberia@smtpl.com

Caution: This product is intended for use by or under the direction of a physician. Prior to use, refer to the "Instructions for use" supplied with these devices for indications, contraindications, side effects, suggested procedure warnings and precautions. As part of our continuous product development policy we reserve the right to change product specifications without prior notification. Check the regulatory status of the device before distribution in areas where CE marking is not the regulation in force. Illustrations are artist's representations only and should not be considered as engineering drawings or photographs. Photos on file at Sahajanand Medical Technologies Pvt. Ltd. **Disclaimer:** © 2020 Sahajanand Medical Technologies Pvt. Ltd. - All rights reserved. Specifications are subject to modification, revision and improvement.



